




# Stuðningshandfang á rúm frá HMN

## Notandaleiðbeiningar

Trönuhrauni 8  
220 Hafnarfjörður  
Sími 565 2885 / Fax 565 1423  
[stod@stod.is](mailto:stod@stod.is) / [www.stod.is](http://www.stod.is)


# Stuðningshandfang á rúm frá HMN

Handfangið skal staðsetja þannig að minnst 6 cm eða meira en 25 cm fjarlægð sé frá enda handfangsins að rúmgafla

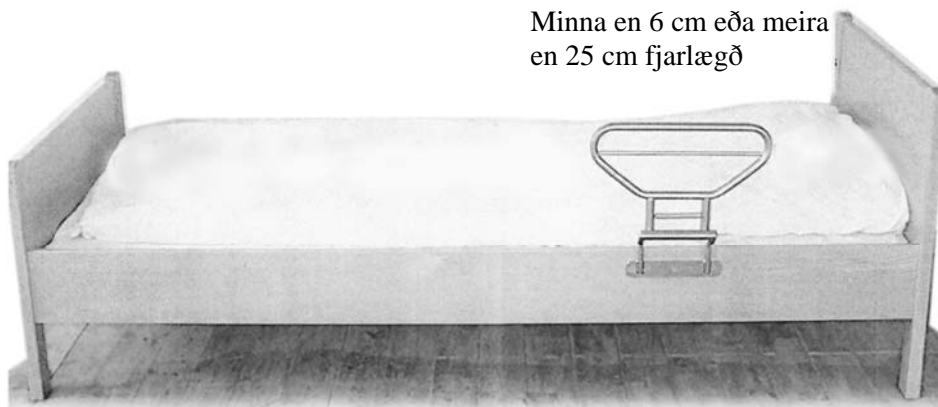

Þegar festingin er staðsett á rúmbríkina þarf að huga að því að handfangið nái að minnsta kosti 22 cm upp yfir dýnuna.

Handfangið er tekið af með því að toga í gripið.

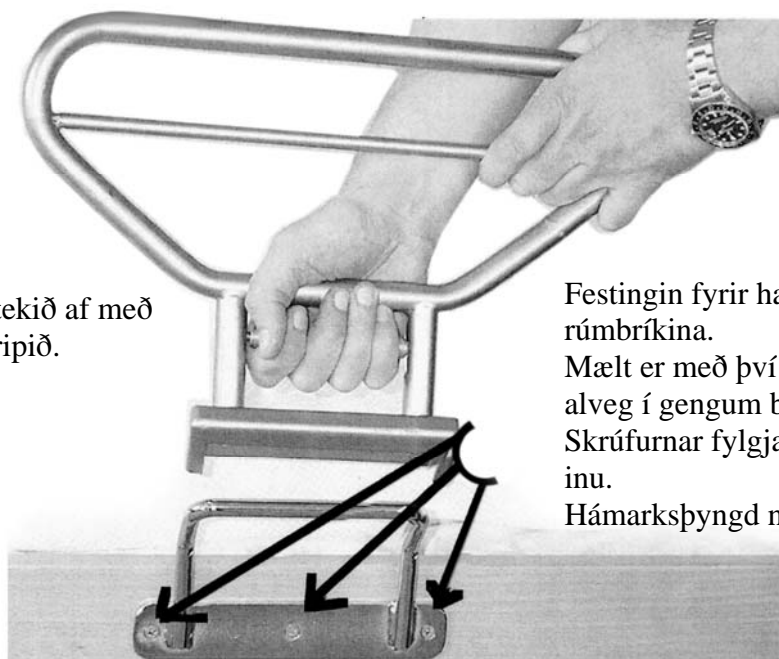

Festingin fyrir handfangið er skrúfuð á rúmbríkina. Mælt er með því að skrúfurnar fari alveg í gengum bríkina. Skrúfurnar fylgja ekki með handfanginu. Hámarksþyngd notanda er 120 kg.

Þrifið með rökum klút og mildum hreinsiefnum, Hámarkshiti 70°C.

HMN a/s , Midtager 22, 2605 Brøndby, Danmark  
Tlf. +45 43.962400 Fax. +45 43.962692 E-mail [info@hmn.dk](mailto:info@hmn.dk)

Artkl. Nr. 221307 <b>Max. users load 120 kg</b> 2 year's complaint right against manufacturing errors	Cleaning: Ordinary cleaning detergents at max. 70 degrees.
	<b>CE</b>
Reserve dele til støttegreb	
Release handle for support handle	Locking taps
Taps for release handle	Feeders for locking taps.
Rubber for release handle	

PRODUCED IN THE EEC